

LOGIN GUIDE


This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare*.




Dexcom

If you are using a Dexcom mobile application, go directly to step 3.

- 1 **Create a Dexcom Clarity account** from this site:
<https://uam2.dexcom.com/index.html#/register>

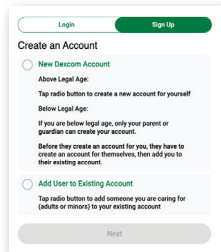


- 2 **Download the Dexcom Clarity Uploader from this page:**
<https://clarity.dexcom.eu/?locale=fr-FR#/home-user-upload>
Then **upload** the monitor data to Dexcom Clarity by following the instructions on the screen.

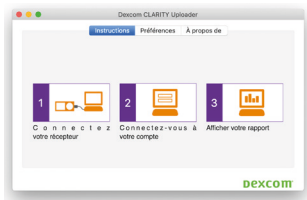



- 3 **Log in to the myDiabby Healthcare account** from a computer **OR** the mobile
- 4 **Click on the "My Connected Objects" tab and then on "Connect my continuous glucose monitor".**



5 Choose the Dexcom button, then click on the **"CONNECT MY HEALTH APPLICATION"** button

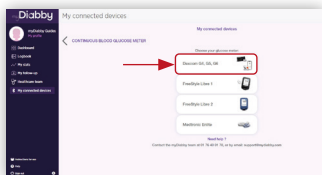


6 Enter your **Dexcom account login details and password**. Read the terms and conditions of use, then **enter your name and click "Allow"** to accept.

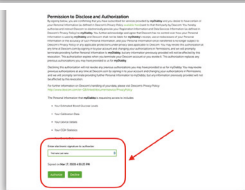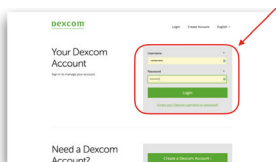

7 This message appears on the screen and confirms that the **Dexcom account is connected to myDiabby Healthcare**. Click on **"Back"** to return to myDiabby Healthcare.

Continuous blood glucose data is now shared on myDiabby Healthcare !

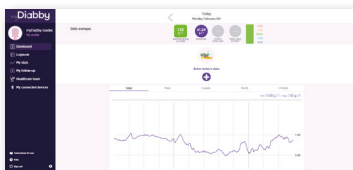

PLEASE NOTE

Upload the Dexcom meter data to Dexcom Clarity regularly so that the healthcare team can see the results on myDiabby Healthcare.

> Once the **Dexcom** and **myDiabby Healthcare** accounts are connected, data is regularly sent automatically. The data displayed is **3 hours behind** the latest available data.

> If you are using a **Dexcom G6** in combination with the **t:slim X2 Tandem Insulin Pump**, you only need to upload the pump to myDiabby Healthcare for the **Tandem Pump + G6 data** to appear on myDiabby Healthcare.

If you have any questions about the myDiabby application: support@mydiabby.com or by phone

France : 01 76 40 01 78

Belgique : 02 320 11 96

*The manual is available for download and consultation on the myDiabby Healthcare platform.



MDHC SAS
66 avenue des Champs Elysées
75008 Paris
FRANCE



0459

GCDCXCV05D20250703EN