

LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare*.




Linking the pump with the myDiabby application

- Download the 2 applications myDiabby and mylife to the phone. Log in to each of the 2 applications.**
The mylife application collects data from the Ypsopump insulin pump. Mylife will then automatically send the data to myDiabby Healthcare, which is the application that connects you with your healthcare team.






- Connect the Ypsopump to the mylife application via Bluetooth following the insulin pump guide.**


- Log in to the myDiabby application and open the "My connected objects" tab**

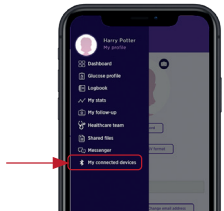


- Select "Connect my insulin pump", then "Ypsopump". Click on the "Connect my health application" button.**

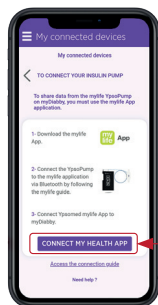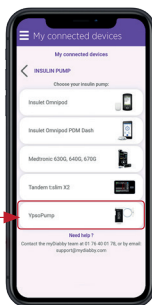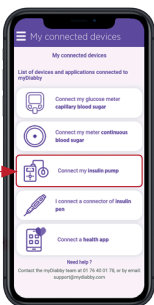

5 Enter your mylife App account login details and confirm

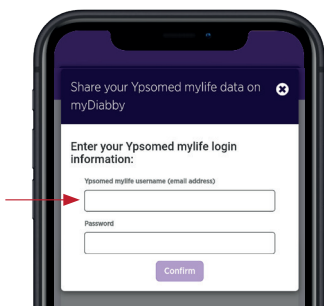

6 The data from mylife Ypsopump is now shared on the myDiabby application!

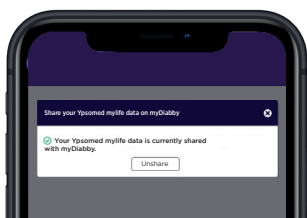

PLEASE NOTE

- All data entered manually or via Bluetooth in the mylife application is shared with the myDiabby application (pump data, blood sugar levels, meals, etc.).
- Once the Mylife and myDiabby Healthcare accounts are connected, data is regularly sent automatically. This sending may cause a delay between the synchronization of data on Mylife and their display on myDiabby Healthcare.



If you have any questions about the myDiabby application:
support@mydiabby.com or by phone

France : 01 76 40 01 78
 Belgique : 02 320 11 96

*The manual is available for download and consultation on the myDiabby Healthcare platform.



MDHC SAS
 66 avenue des Champs Elysées
 75008 Paris
 FRANCE

