



## LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare\*.

## Sharing pump data with the myDiabby application with a phone

### Equipment needed

The **Android smartphone** on which the **myDiabby application** has been downloaded, and patient account **logged in**.



The Omnipod PDM.

An adapter to connect a USB cable to the smartphone.



The Omnipod PDM USB cable.

### How to proceed

- 1 Connect in this order: 1 - the cable to the phone with the adapter
- 2 - the cable to the PDM



- 2 The PDM turns on and emits a beep. It is preparing the data for transfer. Please wait.



- 3 The PDM emits 4 times 2 beeps. It is ready to transfer data to the phone.



- 4 Unlock the smartphone screen. Launch the myDiabby application and data sharing is initiated. The cable symbol flashes and a loading bar appears during data sharing. Please wait.



**Caution:**

Do not disconnect the cable from the meter or phone during sharing data. If there is a lot of data to share, the transfer may take several minutes.

- 5 The pump data is displayed on myDiabby! You can now disconnect the phone and the PDM.

# Sharing pump data with the myDiabby application with a computer


- 1 Download **myDiabby Uploader** for Windows or Mac at [www.mydiabby.com/uploader](http://www.mydiabby.com/uploader).  
Install **myDiabby Uploader** on your computer.



**myDiABBY**  
UPLOADER

MyDiabby Uploader is a small software program that you can install on your computer to **upload data from blood glucose meters, blood glucose monitors and insulin pumps to your myDiabby Healthcare account.**


- 2 **Log in with your usual myDiabby Healthcare login details** and follow the software instructions.

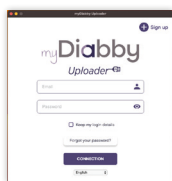


- 3 **Select the device(s) you wish to download and Confirm**



- 4 **Connect the PDM, wait until you hear 2 X 4 "beeps", then click on the "Send" button**

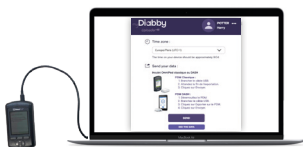

- 5 **The data will now load on myDiabby.** Once the loading is complete, to access the data on myDiabby, click on the **"View Data"** button

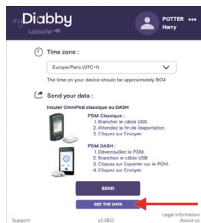

- 6 **The results are now available on myDiabby Healthcare** (basal, bolus rates, blood sugar levels, etc.)

If you have any questions about the myDiabby application:  
[support@mydiabby.com](mailto:support@mydiabby.com) or by phone

France : 01 76 40 01 78

Belgique : 02 320 11 96

MDHC SAS  
66 avenue des Champs Elysées  
75008 Paris  
FRANCE

\*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459

Document applicable from version 2.19 of myDiabby Healthcare

GCOMNV02D20241601EN