

LOGIN GUIDE



This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare*.



NFC connection for Glucofix® Tech and Glucofix® Tech 2K meters

Data transfer is done using the NFC antenna by bringing the meter close to your smartphone.

- 1 Make sure the meter is set to the correct date and time to use connectivity with the myDiabby application.
- 2 **Enable NFC on your smartphone** (in the settings)
on some smartphones, NFC is enabled by default and cannot be disabled
- 3 Log in to the myDiabby application and open the **"My connected objects"** tab
- 4 Select **"Connect my capillary blood glucose meter"**, then select your meter from the list

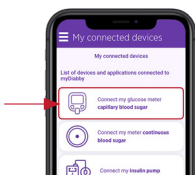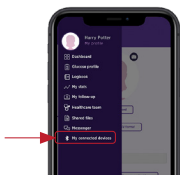

- 5 Click on the **"Connect my meter"** button



- 6 Activate NFC on the meter. To do this:

- 1 Open the player's parameter menu: press the **M (on/off)** and **▲** buttons simultaneously for 4 seconds
- 2 Press the **▼** button, an N will flash
- 3 Press the **M** button, "Off" appears
- 4 Press the **▼** button, "On" appears
- 5 Press the **M** button, "SET" appears, the NFC is activated on your player.
- 6 Turn off the meter: press the M button for 3 seconds



7 The search is underway:

- On Android, **bring the meter close to your smartphone at the NFC spot.**
- On iPhone, **bring your insulin pen close to the NFC spot.** Your iPhone will vibrate once the meter is detected and a confirmation will appear on the screen.

Daily use



1 **Test your blood sugar level** with the meter as usual.

2 Log in to the myDiabby application and open the **"My connected objects"** tab

3 At the meter you have connected, press **"Synchronize"**.


4 **The search is underway:**

- On Android, bring the meter close to your smartphone at the NFC spot.
- On iPhone, bring your insulin pen close to the NFC spot. Your iPhone will vibrate once the meter is detected and a confirmation will appear on the screen.

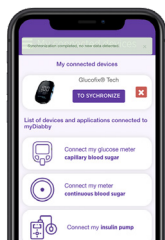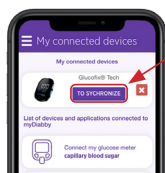

If you have gestational diabetes:

If you have gestational diabetes: once the blood sugar level appears on the myDiabby application, **CHECK** or **VERIFY** the time of day so that the result is in the right place in the logbook. You can **add comments** or insulin doses.

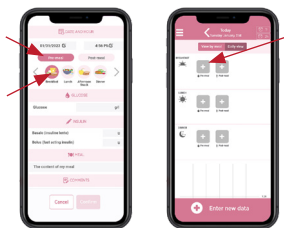

If you have any questions about the myDiabby application:
support@mydiabby.com or by phone

France : 01 76 40 01 78

Belgique : 02 320 11 96



MDHC SAS
66 avenue des Champs Elysées
75008 Paris
FRANCE

*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459

Document applicable from version 2.19 of myDiabby Healthcare