

## LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare\*.





### ACCU-CHEK®

## Linking the meter with the myDiabby application

Make sure the meter is set to the correct date and time to use connectivity with the myDiabby application.

- 1 Write down the serial number of the Bluetooth adapter on a piece of paper
- 2 Enable Bluetooth on the smartphone
- 3 Log in to the **myDiabby application** and open the **"My connected objects"** tab
- 4 Select **"Connect my capillary blood glucose meter"**, then select your meter from the list

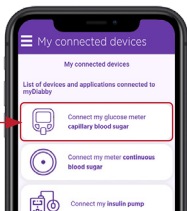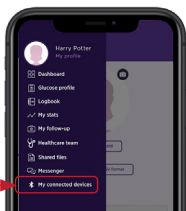

- 5 Click on the **"Connect my meter"** button



- 6 Connect the Bluetooth adapter to the Accu-Chek Mobile blood glucose meter



- 7 Turn on the Bluetooth adapter by pressing the side button. This turns on the meter and puts it in "pairing mode", and the adapter flashes green.



8 On the myDiabby application, click on the "Find my meter" button




9 Enter the 6-digit serial number of the adapter (which you have written down) on the application and confirm.



10 The meter appears on the screen. Click on "Add".

Your meter is now linked to your myDiabby application!

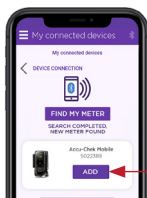


## Daily use

- 1 Test your blood sugar level with the meter as usual.
- 2 Once the blood sugar result appears on the meter screen, **remove the strip from the meter**.
- 3 Open the myDiabby application near the meter.  
*(Make sure Bluetooth is enabled on the smartphone).*
- 4 The blood sugar result is automatically sent to the myDiabby application!



### If you have gestational diabetes:

If you have gestational diabetes: once the blood sugar level appears on the myDiabby application, **CHECK** or **VERIFY** the time of day so that the result is in the right place in the logbook. You can **add comments** or insulin doses.

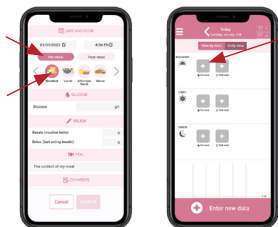

GCACCMV02D20241601EN

If you have any questions about the myDiabby application:  
[support@mydiabby.com](mailto:support@mydiabby.com) or by phone

France : 01 76 40 01 78

Belgique : 02 320 11 96



MDHC SAS  
66 avenue des Champs Elysées  
75008 Paris  
FRANCE

\*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459