

LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare*.




Contour
next ONE
Blood Glucose
Monitoring System

Linking the meter with the myDiabby application

1 Make sure the meter is set to the correct date and time to use connectivity with the myDiabby application.

2 **Enable Bluetooth** on the smartphone.

3 Log in to the **myDiabby application** and open the **"My connected objects"** tab

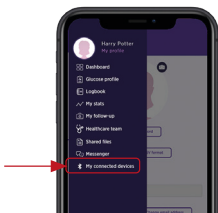

4 **Enable Bluetooth on your Contour next ONE Blood Glucose Meter:** to do this, **press and hold the OK button** until the **BLUE light on the meter flashes** and the Bluetooth symbol ⓧ on the meter screen goes on.




5 Click on the **"Connect my meter"** button, then on **"Find my meter"** to start the pairing.



6 The meter appears on the screen. Click on **"Add"**.

Your meter is now linked to your myDiabby application!

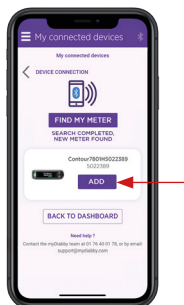

Daily use

- 1 **Test your blood sugar level** with the meter as usual.




- 2 When the blood sugar result is shown on the meter's screen, the Bluetooth symbol indicates that the meter is ready to send the result to the myDiabby application.

- 3 Make sure that **Bluetooth is enabled on the smartphone**, and **open the myDiabby application**, next to the meter still on.


- 4 **The blood sugar result is automatically sent to the myDiabby application!**

If you need to change the meal or time of day for this blood sugar level, you can click on it and enter the relevant information.

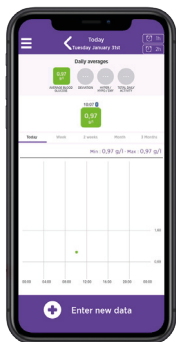

If you have gestational diabetes:

If you have gestational diabetes: once the blood sugar level appears on the myDiabby application, **CHECK** or **VERIFY** the time of day so that the result is in the right place in the logbook. You can **add comments** or insulin doses.

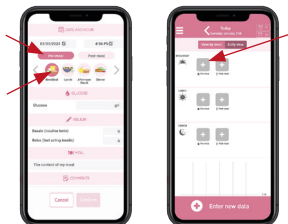

If you have any questions about the myDiabby application:
support@mydiabby.com or by phone

France : 01 76 40 01 78

Belgique : 02 320 11 96



MDHC SAS
66 avenue des Champs Elysées
75008 Paris
FRANCE

*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459