

LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare*.



Sharing pump data with the myDiabby application with a phone

Equipment needed

The **Android smartphone** on which the **myDiabby application** has been downloaded, and patient account **logged in**.



The Omnipod Dash PDM.


An adapter to connect a USB cable to the smartphone.



The Omnipod Dash PDM USB cable.

How to proceed

- 1 Open the **myDiabby application** on your phone and log in with your login details
- 2 First connect the cable to the phone with the adapter
- 3 **Unlock your PDM Dash** (with your PIN code). Then **connect the cable to the PDM Dash**
- 4 The PDM displays the message «Export PDM data» or "Load PDM". > Click on "EXPORT". Wait for "Preparing files for export".
- 5 The PDM says "Your PDM data is ready to be exported". Click on "Done".
 - The PDM is ready to transfer the data to the phone.
 - The cable symbol flashes and a loading bar appears during data sharing. **Please wait.**

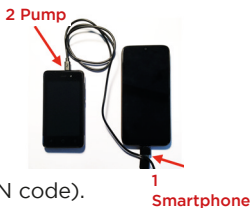

Caution:

Do not disconnect the cable from the meter or phone during sharing data. If there is a lot of data to share, the transfer may take several minutes.



- 6 The pump data is displayed on myDiabby! You can now disconnect the phone and the PDM.

Sharing pump data with the myDiabby application with a computer

- 1 Download **myDiabby Uploader** for Windows or Mac at www.mydiabby.com/uploader.
Install **myDiabby Uploader** on your computer.



myDiABBY
UPLOADER

MyDiabby Uploader is a small software program that you can install on your computer to **upload data from blood glucose meters, blood glucose monitors and insulin pumps to your myDiabby Healthcare account.**

- 2 **Log in with your usual myDiabby Healthcare login details** and follow the software instructions.



- 3 **Select the device(s) you wish to download and Confirm**


- 4 **Unlock the PDM Dash with the pin code, then connect the PDM Dash to the computer with a USB cable.**



- 5 **On the PDM Dash, choose "Export Data".** Wait for the data export to complete and then press OK. Click on **"Send"** on the myDiabby Uploader.



- 6 **The data will now load on myDiabby.** Once the loading is complete, to access the data on myDiabby, click on the **"View Data"** button.

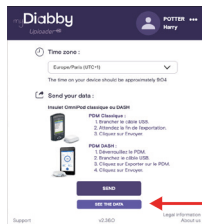

- 7 **The results are now available on myDiabby Healthcare** (basal, bolus rates, blood sugar levels, etc.)

If you have any questions about the myDiabby application:
support@mydiabby.com or by phone

France : 01 76 40 01 78
Belgique : 02 320 11 96

MDHC SAS
66 avenue des Champs Elysées
75008 Paris
FRANCE

*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459