





ACCU-CHEK®

LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare*.

Linking the meter with the myDiabby application

- 1 Make sure the meter is set to the correct date and time to use connectivity with the myDiabby application.
- 2 **Enable Bluetooth** on the smartphone
- 3 Log in to the **myDiabby application** and open the **"My connected objects"** tab
- 4 Select **"Connect my capillary blood glucose meter"**, then select your meter from the list

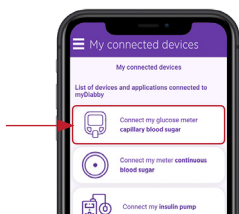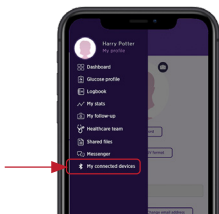


- 5 Click on the **"Connect my meter"** button



- 6 Put the meter in pairing mode. To do this


- Turn on the meter with the **OK** button
- In the meter menu, go to **"Settings"** and then **"Wireless"**
- When asked "No devices linked. Do you want to establish a connection?" Answer **YES**
- A **6-digit pin code** appears on the meter's screen.

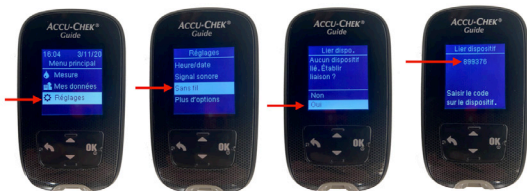


- 7 On the myDiabby application, click on the "Find my meter" button



- 8 Enter the 6-digit code that appears on the meter's screen in the application, then confirm.

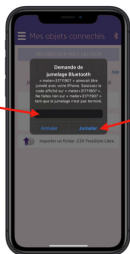

- 9 The meter appears on the screen. Click on "Add".

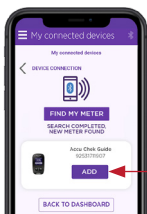


Daily use

- 1 Test your blood sugar level with the meter as usual.
- 2 Once the blood sugar result appears on the meter screen, **remove the strip from the meter**.
- 3 **Open the myDiabby application** near the meter.
(Make sure Bluetooth is enaled on the smartphone).
- 4 **The blood sugar result is automatically sent to the myDiabby application!**



If you have gestational diabetes:

If you have gestational diabetes: once the blood sugar level appears on the myDiabby application, **CHECK** or **VERIFY** the time of day so that the result is in the right place in the logbook. You can **add comments** or insulin doses.

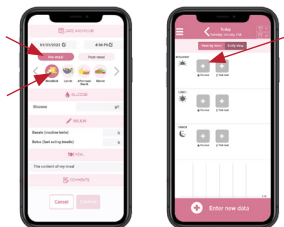

If you have any questions about the myDiabby application:
support@mydiabby.com or by phone

France : 01 76 40 01 78

Belgique : 02 320 11 96



MDHC SAS
66 avenue des Champs Elysées
75008 Paris
FRANCE

*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459