

## LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare\*.



FreeStyle Libre 

If you are using the FreeStyle LibreLink or LibreView application, go directly to step 3.

- 1** Create a LibreView account here: [www.libreview.com/auth/register](http://www.libreview.com/auth/register)
- 2** Upload data from your FreeStyle Libre player to your LibreView account using a computer and the yellow USB cable supplied with the player. For help downloading data to LibreView, go to : [pat.libreview.io/support](http://pat.libreview.io/support)
- 3** Log in to the myDiabby application and open the «My connected objects» tab
- 4** Click on the tab «My Connected Objects» tab, then on «I'm connecting a health application».



- 5** Select the button **FreeStyle Libre**


FreeStyle Libre 

## 6 Enter your **FreeStyle LibreLink / LibreView** account **username and password**.


Read the conditions of use, then click «**Authorise**» to accept data sharing.

## 7 This message appears on the screen and confirms that the **LibreView account is connected to myDiabby Healthcare**.

Click on «**Back**» to return to myDiabby Healthcare.

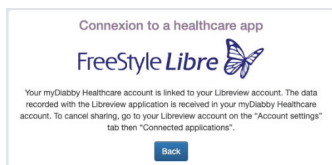

## Your continuous glucose data are now shared on myDiabby Healthcare!

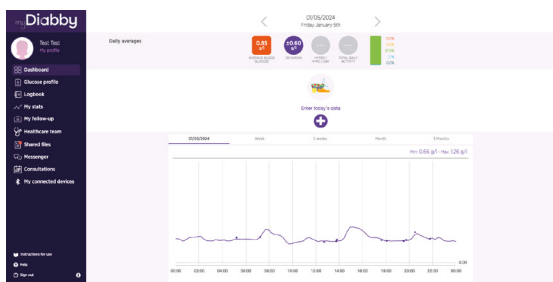

### PLEASE NOTE

Regularly upload data from the **FreeStyle Libre** sensor to **LibreView** so that the medical team can see the results on myDiabby Healthcare.

>Once LibreView and myDiabby are connected, **LibreView automatically sends continuous glucose data to myDiabby Healthcare as soon as it becomes available**. This may cause a delay of a few minutes between loading the data onto LibreView and displaying it on myDiabby Healthcare.

> If you would like to retrieve data from your FreeStyle Libre meter on myDiabby Healthcare using another method, please consult our help centre: [help.diabby.com](http://help.diabby.com)

Good use document for patients using these medical devices. Please refer to the user manuals for FreeStyle Libre 2, FreeStyle LibreLink and LibreView: <https://www.diabetescare.abbott/support/manuals/fr.html>

The sensor housing, FreeStyle, Libre and associated marks are trademarks of Abbott and used with permission.

If you have any questions about the myDiabby application:  
**support@mydiabby.com** or by phone

France : **01 76 40 01 78**

Belgique : **02 320 11 96**



MDHC SAS  
66 avenue des Champs Elysées  
75008 Paris  
FRANCE

\*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459