



## LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare\*.



## Linking the meter with the myDiabby application

- 1 Make sure the meter is set to the correct date and time to use connectivity with the myDiabby application.
- 2 **Enable Bluetooth** on the smartphone
- 3 Log in to the **myDiabby application** and open the **"My connected objects"** tab
- 4 Select **"Connect my capillary blood glucose meter"**, then select your meter from the list

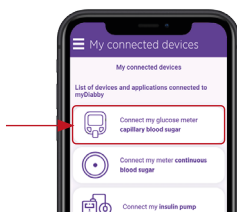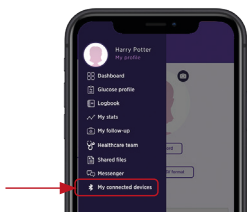

- 5 Click on the **"Connect my meter"** button



- 6 Put the meter in pairing mode. To do this:

- Turn on the meter using the **S** button
- Press and hold the S button until you see **"SET"** appear on the meter's screen.
- Press the button ► to see **"Yes"** at the bottom left of the screen, then press the **S** button
- **"bT"** appears on the meter's screen. Press ► 3 times to see **"PAir"** appear on the bottom right of the meter's screen. Then **press the S** button.
- The word **"Pin"** appears on the meter's screen




- 7 On the myDiabby application, click on the "Find my meter" button



- 8 Enter the 6-digit code that appears on the meter's screen in the application, then confirm.



- 9 The meter appears on the screen. Click on "Add".

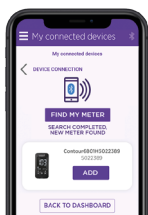


## Daily use

- 1 Test your blood sugar level with the meter as usual.
- 2 Once the blood sugar result appears on the meter screen, **remove the strip from the meter**.
- 3 Open the myDiabby application near the meter. *(Make sure Bluetooth is enabled on the smartphone).*
- 4 The blood sugar result is automatically sent to the myDiabby application!



### If you have gestational diabetes:

If you have gestational diabetes: once the blood sugar level appears on the myDiabby application, **CHECK** or **VERIFY** the time of day so that the result is in the right place in the logbook. You can **add comments** or insulin doses.

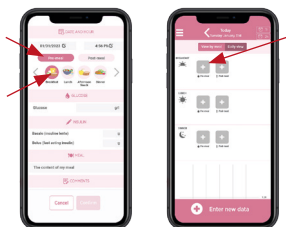

If you have any questions about the myDiabby application:  
[support@mydiabby.com](mailto:support@mydiabby.com) or by phone

France : 01 76 40 01 78  
 Belgique : 02 320 11 96



MDHC SAS  
 66 avenue des Champs Elysées  
 75008 Paris  
 FRANCE

\*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459