

LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare*.




Medtronic 640G

Sharing via USB cable on a computer or Mac


Phase 1: myDiabby Uploader

- 1 Download myDiabby Uploader for Windows or Mac at www.mydiabby.com/uploader. Install myDiabby Uploader on your computer.

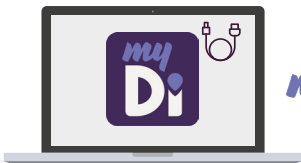


myDiABBY
UPLOADER

- 2 Log in with your usual myDiabby Healthcare login details and follow the software instructions.

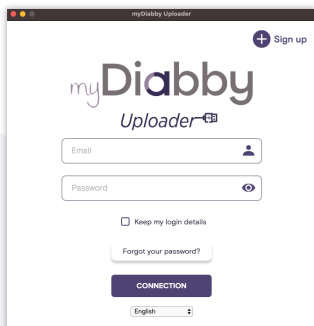


- 3 Select the device(s) you wish to download and confirm.

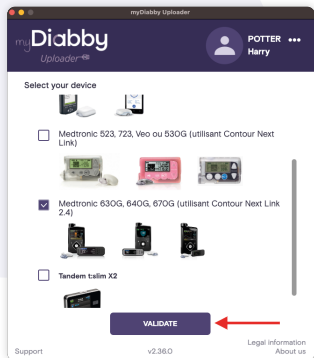

Phase 2: the connection


- 4 If the insulin pump and Contour Next Link 2.4 meter are connected, check the box **"The meter and pump are linked"**.

If the insulin pump and the Contour Next Link 2.4 meter are not connected, uncheck the box **"The meter and the pump are linked"** and enter the pump serial number (on the back of the pump)

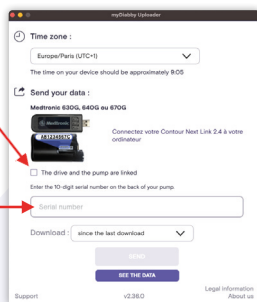

- 5 Connect the Contour Next Link 2.4 reader to a USB port on the computer.

- 6 Select the duration of the data to be uploaded, then click on **"Send"**.

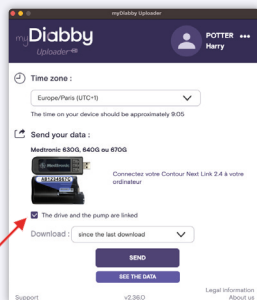

- 7 Your data uploads to the computer. If the pump was not connected to the meter, you will need to confirm the connection on the pump to start the upload.

- 8 At the end of the upload, click on **"View Data"** to go to the myDiabby online platform and see all your results.

If you have any questions about the myDiabby application:
support@mydiabby.com or by phone

France : 01 76 40 01 78

Belgique : 02 320 11 96



MDHC SAS
66 avenue des Champs Elysées
75008 Paris
FRANCE

*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459