

LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare*.



Sharing pump data with the myDiabby application with a phone

Equipment needed

The **Android smartphone** on which the **myDiabby application** has been downloaded, and patient account **logged in**.



The Tandem t:slim x2 pump.

An adapter to connect a USB cable to the smartphone.



The Tandem t:slim x2 pump USB cable.

How to proceed

1 First connect **the cable to the phone** with the adapter


2 Connect **the cable to the pump**.

3 The **myDiabby application opens directly** and data sharing starts on the application.

If the application asks for permission to access the phone's USB, click "Allow", or "Accept".

4 The **cable symbol flashes** and a progress bar is displayed during data sharing. **Wait** until the transfer is complete.

5 The **pump data is displayed on myDiabby!** You can now disconnect the phone and the pump.

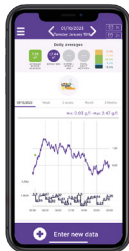

Sharing pump data with the myDiabby application with a computer


- 1 Download **myDiabby Uploader** for Windows or Mac at www.mydiabby.com/uploader



myDiABBY
UPLOADER

MyDiabby Uploader is a small software program that you can install on your computer to **upload data from blood glucose meters, blood glucose monitors and insulin pumps to your myDiabby Healthcare account.**

- 2 Install **myDiabby Uploader** on your computer, then **Log in with your usual myDiabby Healthcare login details** and follow the software instructions.

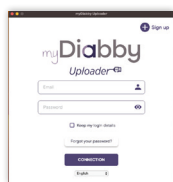

- 3 **Select the devices** you want to download and click on **Confirm**.



- 4 **Connect the Tandem t:slim X2 pump** to the computer with a USB cable.


- 5 Click on **"Send"** on myDiabby Uploader.



- 6 **The data is uploaded to the computer.**

NB > If a Dexcom monitor was connected to the Tandem t:slim X2 pump, the continuous glucose data will also transfer to myDiabby

- 7 At the end of the upload, click on **"View Data"** to go to the myDiabby online platform and see all your results.

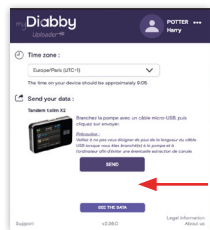

If you have any questions about the myDiabby application:
support@mydiabby.com or by phone

France : 01 76 40 01 78

Belgique : 02 320 11 96

MDHC SAS
66 avenue des Champs Elysées
75008 Paris
FRANCE

*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459

Document applicable from version 2.19 of myDiabby Healthcare

GCTX2V02D20241601EN