

LOGIN GUIDE

This login guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare*.




Contour
next

Linking the meter with the myDiabby application

⌚ Before starting, make sure the meter is set to the **same date and time as the phone**. If not, set it according to the meter's instructions.

1 Enable Bluetooth on the smartphone.

2 Log in to the **myDiabby application** and open the **"My connected objects"** tab

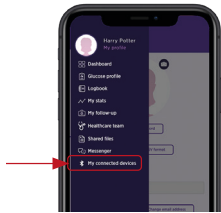


3 Turn on the Contour Next meter by long pressing the OK button.

4 Enable Bluetooth on the meter by long pressing the ▲ button. The Bluetooth logo ⌘ appears on the meter's screen.



5 In the **myDiabby application**, click on the button **"Connect my capillary blood glucose meter"** and then **"ASCENSIA Contour NEXT"**.

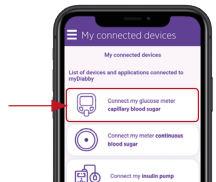

6 Click on the **"Connect my meter"** button.




7 Enter the 6-digit code that appears on the meter's screen in the application, then confirm.



- 8 The meter appears on the screen. Click on **"Add"**.

Your meter is now linked to your myDiabby application!

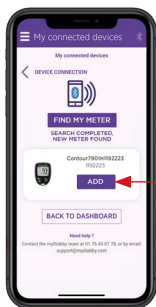


Daily use

- 1 Test your blood sugar level with the meter as usual.
- 2 Once the blood sugar result appears on the meter screen, remove the strip from the meter.



- 3 The blood sugar result is automatically sent to the myDiabby application!

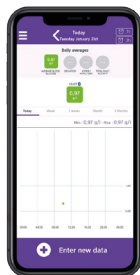


- 4 If you need to change the meal or time of day for this blood sugar level, you can click on it and enter the relevant information.



If you have gestational diabetes:

If you have gestational diabetes: once the blood sugar level appears on the myDiabby application, **CHECK** or **VERIFY** the time of day so that the result is in the right place in the logbook. You can **add comments** or insulin doses.

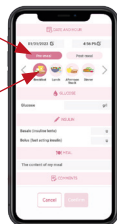

GCCNV02D20241601EN

If you have any questions about the myDiabby application:
support@mydiabby.com or by phone

France : 01 76 40 01 78

Belgique : 02 320 11 96



MDHC SAS
66 avenue des Champs Elysées
75008 Paris
FRANCE

*The manual is available for download and consultation on the myDiabby Healthcare platform.



0459