


3 Open the **myDiabby** application (on smartphone OR computer) and click on «**Enter new data**».



4 Scroll to the bottom of the screen and click on «**Import a .csv file**» then «**Import a CareLink .csv file**».

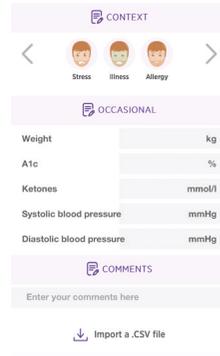

5 Select the **.csv file that has been saved** on your phone or computer from the CareLink site

6 Results from the **MiniMed 780G pump and its sensor** are now shared on the **myDiabby application**.



#### NOTES

All CareLink data is shared on myDiabby Healthcare when it is shared: continuous glucose data, capillary blood glucose levels, Basal, Bolus, carbohydrates, pump information, etc.

# myDiabby

HEALTHCARE

## CONNECTION GUIDE

This connection guide provides the specific login information for your device. Please refer to the manual for all information on using myDiabby Healthcare\*.



### Medtronic MiniMed 780G

## Phase 1 : Put pump data on the CareLink platform

If you use your pump with the **MiniMed Mobile** application, your data is automatically saved on **CareLink**.



Medtronic

If you are not using the application, **download the pump data from CareLink using the reader associated with the pump**. More information at: [carelink.minimed.eu/app/install-uploader](http://carelink.minimed.eu/app/install-uploader)



## Phase 2 : Sharing CareLink data on myDiabby Healthcare

GCM780V02D20241601EN

If you have any questions about the myDiabby application: [support@mydiabby.com](mailto:support@mydiabby.com) or by phone

France : 01 76 40 01 78 Belgique : 02 320 11 96

\*The manual is available for download and consultation on the myDiabby Healthcare platform.




MDHC SAS  
66 avenue des Champs Elysées  
75008 Paris  
FRANCE

## Method 1: Connecting your Carelink account to myDiabby Healthcare account

1 Open the **myDiabby** application (on smartphone OR computer) and click on the «**My Connected Devices**» tab.

2 Click on the «**Connect a health app**» category

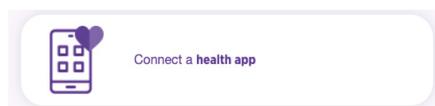

3 From the list of compatible applications, select «**CareLink**».



4 Indicate your **CareLink identifier** in the dedicated pop-up window. This is the same login you use to connect to **CareLink** or the **MiniMed Mobile** application.

Share your CareLink data on myDiabby Healthcare

Enter your CareLink username:

This is the identifier you use on CareLink or the MiniMed Mobile application

Confirm

5 Go to your **CareLink account** to accept the sharing by clicking on the link in the pop-up or via the automatic e-mail you have received.

### Clinic Sharing

New requests:

myDiabby Healthcare Pending request

France

75008, FR


Patient Name: test en

✓ Approve
✗ Reject

Your medical provider has requested to link to your CareLink™ Personal account. When the linking process is complete, your medical provider will be able to see data from your linked medical devices that have been uploaded into your CareLink™ Personal account outside of your face to face consultations.

6 Once you've accepted the sharing, **go back to your myDiabby Healthcare account and the data will now be shared automatically!**

It may take a few minutes for the data to be recovered when you first connect.

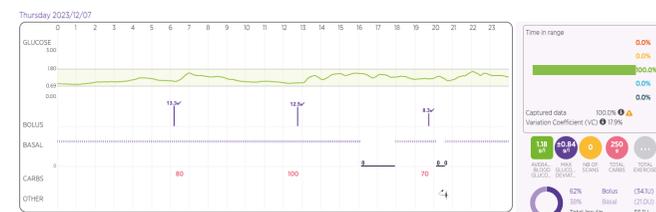

### NOTES

All CareLink data is shared on myDiabby Healthcare when it is shared: continuous glucose data, capillary blood glucose levels, Basal, Bolus, carbohydrates, pump information, etc.

## Method 2 : Recover data manually from CareLink

From the web browser on a computer OR a smartphone:

1 Go to [carelink.minimed.com](http://carelink.minimed.com) and **connect** to the patient account using the same identifiers as for the **MiniMed Mobile** application, or the **CareLink identifiers** directly.

Log in to CareLink™ PERSONAL

Country or Region: France

Language: English

Continue

Forgot password?

or

Create an Account

2 On the CareLink website, click on :  
- The «**Reports**» tab at the top left (curve icon)

- Then, at the bottom of the page, «**Export data (CSV)**»

> A .csv file containing the pump results is saved on the phone or computer.

